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ESPAÑOL

RESPIRONICS®
Envisioning tomorrow. Improving today.™

Manufactured in China by
Respironics (UK) Ltd for

Respironics New Jersey, Inc
5 Wood Hollow Road,
Parsippany, NJ 07054

www.respironics.com

© 2008 Respironics Limited ALL RIGHTS RESERVED. Respironics and SideStream™ are trademarks of Respironics Ltd or companies within the group respectively, registered in the UK and other countries.

1052870 REV2 US 5533501 08/08 BP10717

RESPIRONICS®

LATEX


Reorder Number: **HS800**

NDC 08373-0008-00

Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician or licensed healthcare professional

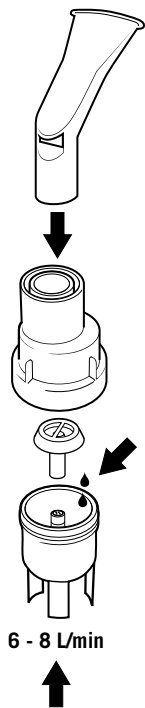3 83730 00800 4

CONTAINS ONE DISPOSABLE SIDESTREAM NEBULIZER, ANGLED MOUTHPIECE & TUBING.

DISPOSABLE: For single patient use.

Not intended for reprocessing. It is recommended that you replace this product after two weeks of use.

INSTRUCTIONS FOR USE:

NOTE: These instructions are general guidelines intended for use by patients under the direction of a licensed healthcare professional.

- 1 Place medication into the medicine cup, 10cc maximum. Screw the nebulizer lid onto the medicine cup.
- 2 Insert the angled mouthpiece into the nebulizer.
- 3 Attach one end of the tubing to the compressor or air source and connect the other end to the nebulizer.
- 4 Switch on compressor or adjust air source to 6 – 8 lpm. Check for air flow through the device.

WARNING: Be sure all connections are adequately secure.

Cleaning: After each use, disconnect tubing from SideStream and wash all parts separately in mild liquid detergent and warm water. Rinse thoroughly and shake out excess water. Allow to air dry and reassemble.

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